



**AED LOCATOR (E.U.) LTD**  
Swallet Farm  
Old Bristol Rad  
East Harptree  
Bristol BS40 6DQ  
T: +44 (0) 1275 332323 / 333999

E: [enquiries@aedlocator.org](mailto:enquiries@aedlocator.org)  
W: [www.heartsafe.org.uk](http://www.heartsafe.org.uk)

## **EXAMPLE - GUARDIAN INSPECTION FORM**

**LOCATION of Cabinet:**

**CABINET NO:**

Check Result:

Does the cabinet open and lock correctly using the access code?	
A RED flashing light on the LOCK when entering your 4-digit code means CHANGE LOCK BATTERIES! Are your lock batteries OK?	
Does the light inside the cabinet turn on when the door is opened?	
Have you ensured that no items in the cabinet are touching the heater?	
Is the AED in operating mode, ie. Green light flashing or OK/Tick in the display?	
Is the AED Silent? - ie. NOT beeping or making any noise?	
Do the electro pads have more than two months remaining from their expiry date?	
Does the battery have more than two months remaining from its' expiry date?	
Is there a resuscitation kit in the cabinet?	



**HeartSafe®**  
**The UK's leading provider of Public Access Defibrillators (PADs)**  
**solutions, protecting communities, uniquely monitored and**  
**linked with NHS Ambulance Trusts**

Reg No. 06720474. Registered in England. VAT Registration No. 944 9801 88  
HeartSafe® is a Registered Trademark of AED Locator (E.U.) LTD

February 2016