

2x TRAUMA WOUND DRESSINGS
COVER & COMPRESS



2x HAEMOSTATIC DRESSING PACK/PLUG WOUND



FOLLOW STEPS TO REDUCE BLOOD LOSS:

1. Use mobile phone on loud speaker. Talk with 999 Ambulance Service.

2. Open kit fully. Use gloves and contents as required.

3. Identify area of bleed. Use shears to remove clothing around wound.

4. Use TRAUMA WOUND DRESSING.

APPLY DIRECT PRESSURE. Cover and compress.

5. Use • HAEMOSTATIC

DRESSING to pack junction
regions like neck, groin or armpit.

Plug wound. Cover and compress with **TRAUMA WOUND DRESSING**.

6. Use TOURNIQUET on leg or arms. Cover and compress with TRAUMA WOUND DRESSING.

DO NOT remove - TOURNIQUET.

Follow the Ambulance Service instructions throughout procedure.

If head/foot injury, seek advice from Ambulance Service.



REASSURE PATIENT

WAIT FOR MEDICAL HELP



www.bleedkit.co.uk © HeartSafe t/a AED Locator (E.U.) Ltd BS8599-1 Compliant - Critical Injury Pack
DOC:HSBK011A

ENSURE SCENE


CALL 999 FOLLOW STEPS ACT CALMLY

CONTINUE TO SPEAK TO AMBULANCE SERVICE

REASSURE PATIENT

WAIT FOR MEDICAL HELP

1. Use mobile phone on loud speaker. Talk with 999 Ambulance Service.

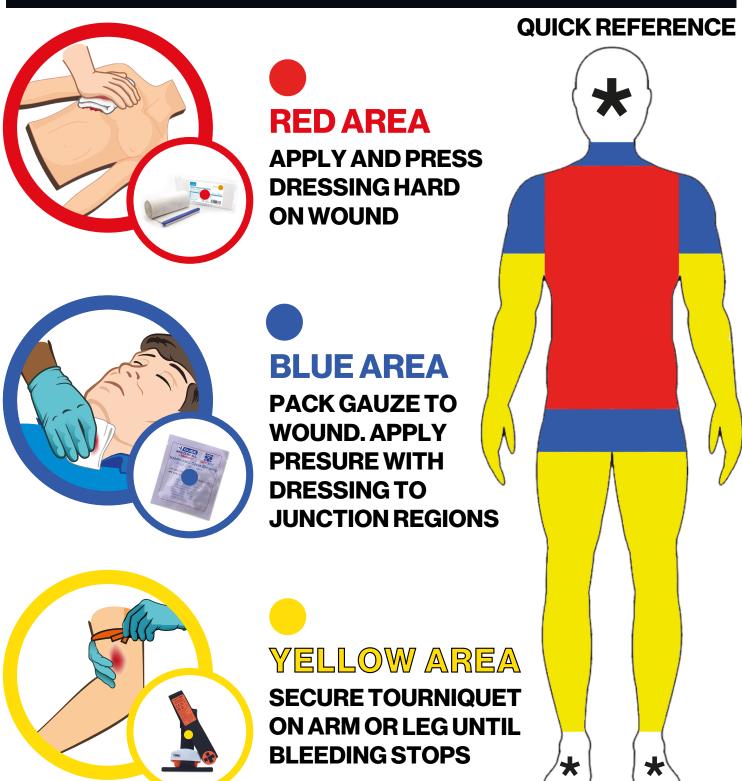

Note: These actions will often cause the patient some pain. The priority is to control and stop the bleeding. The pain will subside and be managed when the ambulance arrives.

Follow the Ambulance Service instructions throughout procedure.